



Patient Initials:	Date of Assessment:
Date of Birth:	Evaluated By:
Gender:	Hospital / City / Country:
Implantation Date:	Date CT / MRI / Slice Thick:

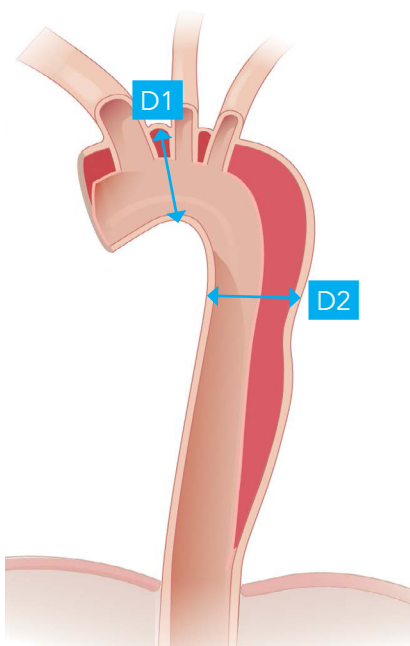

### D1 Example

At the level between the innominate and left common carotid artery

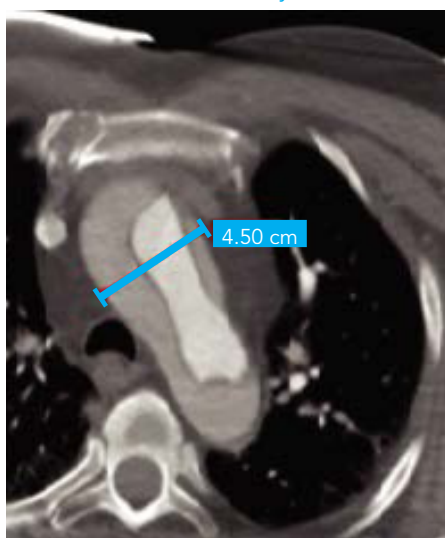

### D2 Example

At the level of the tracheal bifurcation



Using pre-op CT scan, measure the aortic vessel diameter (adventitia to adventitia):

D1	D2	AMDS Configuration
At the level between the innominate and left common carotid artery	At the level of the tracheal bifurcation	AMDS Configuration
20-35mm	20-24mm	AMDS-40-30
20-35mm	25-35mm	AMDS-40-40
36-45mm	27-35mm	AMDS-55-40
36-45mm	36-45mm	AMDS-55-55
Notes:		

\*Ascyrus Medical Dissection Stent Hybrid Prosthesis

CAUTION: Only to be used by trained and certified physicians. Devices not approved for use in all markets.

JOTEC GmbH, Ascyrus Medical, LLC and Ascyrus Medical, GmbH are wholly owned subsidiaries of CryoLife, Inc. © 2020 CryoLife, Inc. All rights reserved.

**Distributor**  
 JOTEC GmbH  
 Lotzenäcker 23  
 72379 Hechingen, Germany  
 Phone: +49 (0)7471 922-0  
 Fax: +49 (0)7471 922-100  
 Email: info.europe@cryolife.com  
 www.jotec.com

**Parent Company**  
 CryoLife, Inc.  
 1655 Roberts Boulevard, NW,  
 Kennesaw, Georgia 30144, USA  
 Phone: +1 (770) 419-3355, +1 (800) 438-8285  
 Fax: +1 (770) 590 3797  
 Email: CustomerService@cryolife.com  
 www.cryolife.com

**Legal Manufacturer**  
 Ascyrus Medical GmbH  
 Bethmannstrasse 8  
 60311 Frankfurt, Germany  
 Email: info@ascyrus.com

JT-SI-0980200-EN V02 10/2020



**CryoLife**  
 Life Restoring Technologies®